



Alarms as Displayed on the Quick Look™ Display

Introduction

This document shows the correlation between how alarms are displayed in the Event List on the Client computer versus how alarms are displayed on the Quick Look Display.



Normal Operation
Date and time are displayed when option is selected



Exit Alarm

(Red Alarm)
Patient transmitter is present
and door is open



Cut Band Alarm

(Red Alarm)
Transmitter opened
If enabled, Global Lockdown causes all
doors to lock and elevator deactivation



Door Alarm

(Red Alarm)
Door opened after Cut Band if Global Lockdown is enabled



Check Band Alarm

(Yellow Alarm)
Loose transmitter



Band Off Alarm

(Red Alarm)
Step up Alarm or transmitter completely off the patient. If enabled, Global Lockdown causes all doors to lock and elevator deactivation



No Signal Alarm

(Red Alarm)
Transmitter is no longer being detected by the system