

for Safe Place[®] and Code Alert[®]

Approved Cleaners

3M [®] Neutral Quat	Hi-Tor [®] Plus
Airx [®] 109A	Isopropyl Alcohol
Cavicide [®] Caviwipes [™] Cavicide1 [™] Caviwipes1 [™]	Oxivir TB PDI [®] Sani-Cloth [®] HB PDI [®] Sani-Cloth Plus PDI [®] Sani-Cloth Prime PDI [®] Super-Sani
Clorox Healthcare [®] Hydrogen Peroxide Cleaner Disinfectants	Quest 256 [®]
Ecolab [®] Quik Fill 920	Steriplex [®] SD RTU
Health-Tec [™]	Virex [®] II-256

Non-Approved Cleaners

(These cleaners may cause plastic embrittlement and erosion)

Bleach Cleaners containing Bleach	PDI [®] Sani-Cloth [®] AF3
Cidex [®] Activated Cidex [®] Plus Cidex [®] OPA	Virex [®] TB Virex [®] HB
3M [™] Phenolic	



Some agents are intended for topical, antimicrobial use only and will not destroy viruses such as Hepatitis B and HIV.

The following equipment and processes must never be used to clean the transmitters:

- Steam autoclave or commercial sterilization procedures using heat, chemical, gas or radiation techniques
- Dishwasher or washing machine
- Soaking or suspending the pendant in water or any other liquids