

Umbilical Cord Transmitters



The Umbilical Cord Transmitter is secured in the infant's umbilical cord clamp and is worn on the umbilical stump.


Transmitters have a 3 month life expectancy. Using a transmitter beyond the printed expiration date can result in system failure, elopement, and/or abduction.



How do I activate the transmitter?

1. With the duck image on the Activation Wand facing upwards, place the tip of the wand on the duck image on the transmitter.
2. Hold the wand on the image for two seconds and then pull upward from the transmitter. A series of green LED blinks indicate the transmitter is active.

NOTE: If needed, the transmitter can be turned off before the green LEDs stop blinking without a transmitter tamper alarm.

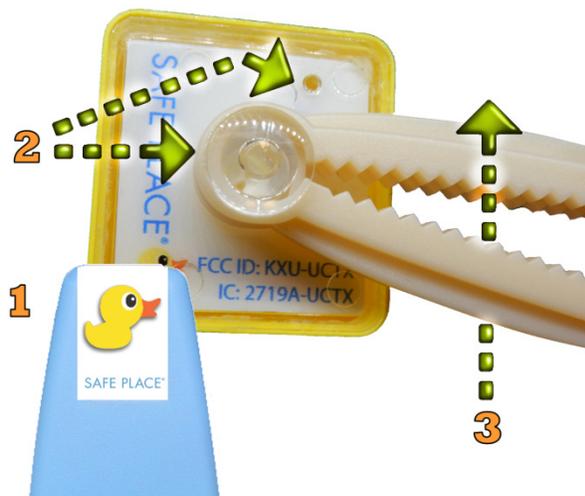

How do I attach the transmitter?

1. Activate the transmitter
2. Place the transmitter post into the clamp on the umbilical stump so that you can see the green LED light.
3. Leave enough length of the umbilical stump beyond the clamp to prevent the transmitter from falling off as the umbilical stump reduces in size.

How do I clean the transmitter?

1. After each use, clean according to your facility's infection control procedures (*use of non-approved cleaners can cause permanent damage to the transmitter and piercing contacts and will void any and all warranties).
2. In addition, scrub each transmitter independently until there are no signs of blood, body fluids, or material on the transmitter.
3. Pay special attention to all surfaces of the transmitter clamp post.
4. Allow transmitters to air dry.

NOTE: The transmitters may be wiped, sprayed or rinsed with water or the approved cleaners during cleaning, but must not be soaked. Soaking transmitters may result in equipment and/or system failure.

How do I discharge and remove the transmitter?

When it is time for an infant to leave your facility, he/she must be discharged from the software (refer to the applicable Series Software User Guide).

1. Check the software to confirm that the transmitter is communicating with the system.
2. Tend to and clear any alarms displayed on the screen for the transmitter.
3. Request a discharge function for the transmitter and complete the action within the allotted time specified by your facility.
4. Using a clamp snipping tool, open the clamp by cutting through the clamp snap.
5. Remove the clamp from the umbilical stump.
6. Remove the transmitter from the clamp and discard the clamp.
7. With the duck image on the Activation Wand facing upwards, place the tip of the wand on the duck image on the transmitter.
8. Hold the wand on the image for two seconds and then pull upward from the transmitter. Three red LED blinks indicate the transmitter has been discharged.
9. Finally, disinfect the transmitter so that it is ready to be used again

* For an up-to-date list of approved cleaning solutions, visit:

[www.rft.com/Resources/Manuals & Specs/Transmitters/Transmitter Approved Cleaning Solutions](http://www.rft.com/Resources/Manuals%20&%20Specs/Transmitters/Transmitter%20Approved%20Cleaning%20Solutions)