


### How do I match the transmitters?

1. Remove the Baby Check Transmitter from its bracket
2. Bring the transmitter 3 to 6 inches from the Infant transmitter and press ✓
3. LED will flash **amber** twice every second as it attempts to match
4. LED will flash **green** three times for a match
5. LED will flash **red** three times for a mismatch. Follow your facility's established procedures for mismatches
6. Repeat for multiple infants
7. Return the Baby Check transmitter to its bracket

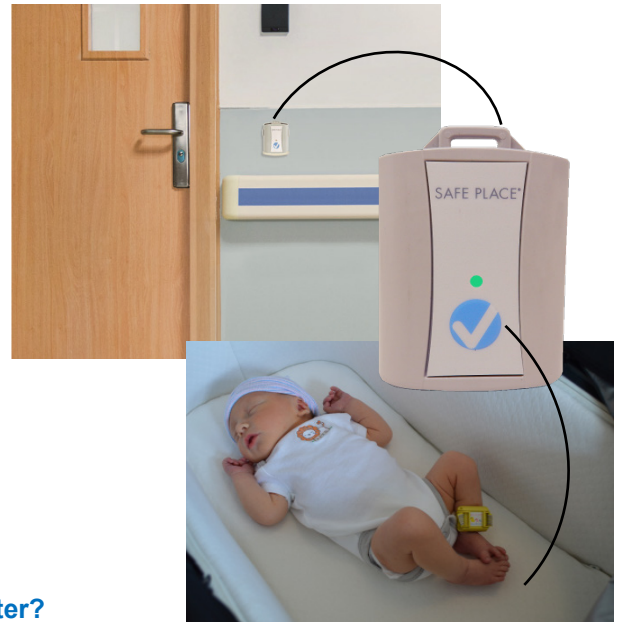

### How do I discharge the transmitter?

1. Check the software and clear any alarms displayed for the transmitter
2. Request a discharge in the software for the Infant transmitter
3. Open the transmitter clamps to remove the banding and discard the used banding material
4. Discharge events should post in the software for both the Baby Check and the Infant transmitters
5. Hold the Clearing Device 3 to 6 inches from the Baby Check transmitter and press ✓
6. Transmitter will flash **green**, **red**, **green**, **red**, then solid **amber** when all transmitters have been disconnected
7. Finally, disinfect the transmitters so they are ready to be used again



### How do I clean the transmitters?

1. Disinfect both the Baby Check and Infant transmitters with an \*approved cleaner
2. Follow the manufacturer's instructions and allow the cleaner to remain on the transmitters for the amount of time specified
3. Transmitters can also be rinsed with water, if required to in the manufacturer's instructions
4. Wipe the transmitters with a clean cloth or towel and allow to air-dry
5. Place the clean transmitters in a storage container for future use

**NOTE:** The transmitters may be wiped, sprayed or rinsed with water or the approved cleaners during cleaning, but must not be soaked. Soaking transmitters may result in equipment and/or system failure.

\* For an up-to-date list of approved cleaning solutions, visit:

**[www.rft.com/Resources/Manuals & Specs/Transmitters/Transmitter Approved Cleaning Solutions](http://www.rft.com/Resources/Manuals%20&%20Specs/Transmitters/Transmitter%20Approved%20Cleaning%20Solutions)**

*Use of non-approved cleaners can cause permanent damage to the transmitter and/or piercing contacts and will void any and all warranties*

3125 North 126<sup>th</sup> Street, Brookfield, WI 53005

**Technical Support:** 800.669.9946, option 5 ([tech@rft.com](mailto:tech@rft.com))

©2020 RF Technologies, Inc. All specifications subject to change without notice.

**Customer Care:** 800.669.9946, option 2 ([customer@rft.com](mailto:customer@rft.com))